DE Körperanalysewaage BS 450

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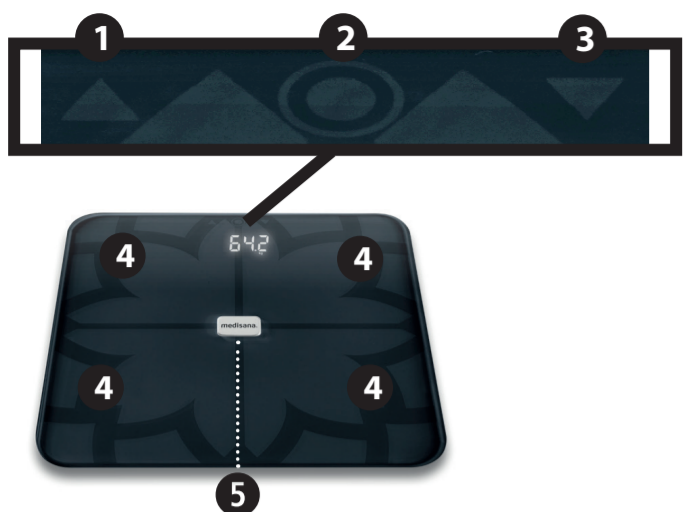
DEUTSCHLAND

GB Body analysis scale BS 450

- 1 ▲ button
- 2 OK button
- 3 ▼ button
- 4 Electrodes
- 5 Battery compartment (on underside)

Explanation of symbols:

This instruction manual belongs to this device. It contains important information about starting up and operation. Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

WARNING!
These warning notes must be observed to prevent any injury to the user.

LOT

LOT number

Manufacturer

40510_40511_BS 450_07/2021_Ver.1.2

We hereby declare that bathroom scales model BS 450, art. 40510/40511 complies with the fundamental requirements of the European RED 2014/53/EU. You can request the complete EU Conformity Declaration from medisana GmbH, Carl-Schurz-Str. 2, 41460 Neuss, Germany, or you can also download it from the medisana homepage (www.medisana.com).

GB Safety Information

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

- These scales are not suitable for people with pacemakers or other medical implants. The body fat information may be inaccurate in people with diabetes or other medical/physical limitations. The same also applies to very highly trained athletes.
- This device is not suitable for pregnant women!
- Any treatment or diet for people who are underweight or overweight requires the qualified advice of a professional (doctor, dietician). The values determined with the scales may be source of useful support.
- Risk of tipping over! Do not place the scale on an uneven surface. Never step onto a corner of the scales. When weighing yourself, do not stand on one side or the edge of the scales.
- Risk of slipping! Do not stand on the scale with wet feet. Do not stand on the scale wearing socks.
- Use the device only according to its intended purpose as specified in the instruction manual. The warranty will be invalidated if the device is used for purposes other than those for which it is intended.
- The scale has been manufactured for domestic use. It is not suitable for commercial use in hospitals or other medical institutions.
- Do not use the device if it does not work properly, if it has been dropped, dropped into water or damaged.
- The scales have a measuring range up to 180kg/396lb/28st 4,8lb. Do not overload the scales.
- Place the scales on a firm, level base. Soft, uneven surfaces are unsuitable for the measuring process and lead to incorrect results.
- Do not use the scale in a location where extremely high temperatures or humidity may occur.
- Keep the scale away from water.
- Handle the scale with care. The scale must not be subjected to impacts or vibration. Do not drop the scale.
- Stand carefully on the scale. Do not jump or hop onto the standing area. This could lead to defects of the weighing mechanism.
- Do not attempt to dismantle the scale, otherwise the warranty will be invalidated. This unit contains no user-maintainable or user-replaceable parts.
- Do not attempt to repair the device yourself in the event of a malfunction since this will invalidate the warranty. Repairs should only be carried out by authorised service centres.

BATTERY SAFETY INSTRUCTIONS

- Do not disassemble batteries!
- Remove discharged batteries from the device immediately! Increased risk of leakage!
- Avoid contact with skin, eyes and mucous membranes! If battery acid comes in contact with any of this parts, rinse the affected area with copious amounts of fresh water and seek medical attention immediately!
- If a battery has been swallowed seek medical attention immediately!
- Insert the batteries correctly, observing the polarity!
- Always keep the battery compartment well closed!
- Remove the batteries from the device if it is not going to be used for an extended period!
- Keep batteries out of children's reach!
- Do not attempt to recharge these batteries! There is a danger of explosion!
- Do not short circuit! There is a danger of explosion!
- Do not throw into a fire! There is a danger of explosion!
- Keep unused batteries in their packaging away from metal objects in order to prevent short circuiting!

Scope of supply

Please check first of all that the unit is complete and is not damaged in any way. If in doubt, do not use the appliance. Send it to a service point. The following parts are included:

- 1 medisana Body analysis scale **BS 450**
- 4 Batteries (Type AAA, 1.5 V)
- 1 Instruction manual

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your dealer without delay.

WARNING
Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!

Connecting to VitaDock+

Free download of the VitaDock+ app in the app store or Google Play store. Activate Bluetooth® on your iOS or Android appliance. Start the app. Touch the Bluetooth® symbol to open the list of devices. Scroll down the device list until you see the scales. Touch the arrow beside the scales to start the setup process.

Initialise the scales

Make sure the scales are on a firm, level surface. If you only want to weigh yourself but have moved the scales just before, you must first initialise the scales. To do this, press your foot on the middle of the surface of the scales. "0.0 kg" appears in the display. When the scales switch off, they are ready for the step-on function. This procedure is not necessary if you have not moved the scales.

Weighing

1. Step onto the scales and stand still. The device switches on automatically and the display briefly shows „0.0 kg“.
2. Your weight is measured, flashes twice and is then finally displayed.
3. Step down from the scales. The scales switch off automatically after 10 seconds.

Weighing and measuring the body analysis values

The scales can store data for up to 9 people: sex, age, height.

1. Press your foot on the middle of the surface of the scales. „0.0 kg“ appears in the display.
2. Press the OK button 2. The last used personal data is flashing in the display.
3. Now press button 1 or button 3 to select the desired memory location (0 - 8) for the user profile. NOTE: If you use the scales in guest mode „P0“, no data is saved. The weight data cannot be transferred by Bluetooth® in this mode.
4. Press the OK button 2 to confirm the selection. Now the symbol for the sex is flashing in the display.
5. Set your sex by pressing button 1 or button 3 to select the appropriate symbol in the display.
6. Press the OK button 2 to confirm the selection. The preset height starts flashing in the display.
7. Select your height using button 1 and button 3.
8. Press the OK button 2 to confirm the selection. The preset age starts flashing in the display.
9. Set your age by pressing button 1 or button 3.
10. Press the OK button 2 to confirm the setting. The preset target weight starts flashing in the display.
11. Select target weight by pressing button 1 or button 3.
12. Press the OK button 2 to confirm the setting.
13. The settings are now concluded. „0.0“ appears in the display.

Step barefoot onto the scales and stand still.

Place your feet on the electrodes 4. First, your weight is displayed, then difference between actual weight and target weight is shown. Next the values for BMI, KCAL, body fat, muscle, water and bone weight are shown one after another in the display. The series of measured values is repeated twice in the display. The scales then switch off automatically. Step off the scales.

Change the unit of weight

Press your foot on the middle of the surface of the scales shortly to activate the device. Then press 1 or 3 to select the unit.

Error messages

O-Ld = The scale is overloaded. **LO** = The battery is low and must be replaced. **ERR2** = Body analysis values cannot be calculated. **P1P2** or other combinations of memory locations = The scale recognizes users already saved based on their weight. In this case, two users have a similar weight. To select the correct user press the UP key for the left value (example "P1") and the DOWN key for the right value (example "P2"). If your personal scales do not function to your satisfaction, check the following before contacting a service centre: • Check whether the batteries have been inserted correctly. • Check whether the correct unit of weight has been selected. • Check whether the scales are free-standing on a solid and even surface. They must not be touching a wall or any other object. • Weigh yourself again.

Care and maintenance

Never use aggressive cleaning agents or stiff brushes. Clean the scale using a soft and lightly moistened cloth. Never use abrasive cleaning agents or alcohol. Do not allow any water to get into the unit. Do not use the unit again until it is completely dry.

Disposal

This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Please remove the battery before disposing of the device/unit. Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop.

Technical specifications

Name and model: medisana body analysis scale BS 450

Voltage supply: 6 V (4x 1.5 V, AAA)

Measuring range: up to 180kg/396lb/28st 4.8lb

Graduation: 0.1 kg/0.2 lb/0.2 lb

Dimensions: 319 x 319 x 23 mm

Weight: 1.93 kg

Item number: 40510/40511

EAN number: 4015588 40510 5/4015588 40511 2

Frequency band: 2400 ~ 2478 MHz

Maximum radio-frequency power transmitted in the frequency band: -3 dBm

In accordance with our policy of continual product improvement, we reserve the right to make technical and optical changes without notice.

The current version of this instruction manual can be found under www.medisana.com

Warranty and repair terms

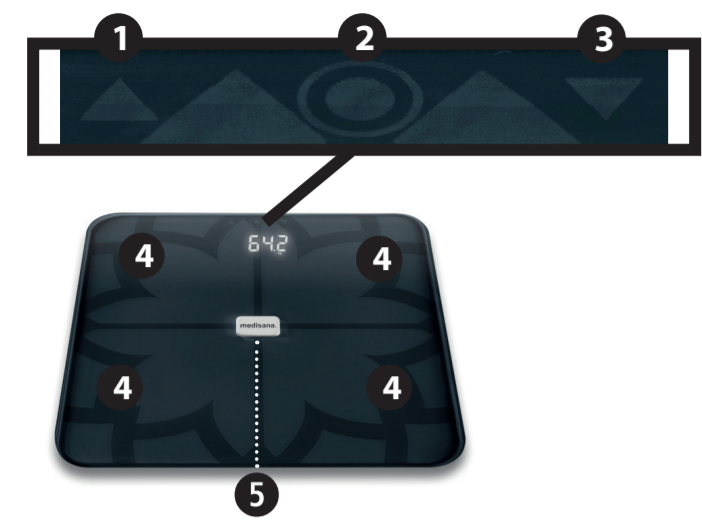
Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is. The following warranty terms apply:

1. The warranty period for medisana products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. nonobservance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear (batteries etc.).
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

medisana GmbH
Carl-Schurz-Str. 2
41460 NEUSS
GERMANY

The service centre address is shown on the attached leaflet.

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