# medisana®

## DE DG 100

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# medisana®

## EN DG 100

- ① on/off button
- ② battery level indicator
- ③ USB port (charge battery)

WARNING! Risk of swallowing by small children! Children are not allowed to play with the device. Keep it away from children.

WARNING! Danger of fire/explosion or chemical burns! The item contains a lithium-ion battery. This must not be removed, taken apart, thrown into a fire or short circuited.

CAUTION! Possible property damage! Do not drop the device and protect it from impact. Do not expose the device to extreme temperatures or extreme temperature fluctuations. Protect the device from direct sunlight and dust. When cleaning, do not use any strong chemicals, or aggressive or foaming cleaning materials. Protect the display from hard, scratching objects.

CAUTION! Do not dispose of the device in the household waste! If you have any questions, please contact your municipal waste disposal authority. Do not dispose of spent batteries in domestic waste, rather in special waste or at a battery collection point at your retailer! The packaging is recyclable or can be recycled into raw materials. Please dispose of unwanted packaging material properly.

Subject to technical changes in the course of constant product development.

Chrono Eyewear BV  
Saal van Zwanenbergweg 11  
5026 RM Tilburg, the Netherlands

The service address can be found on the separate attachment sheet.

## Warning

- Do not use the light glasses longer than intended. In most cases, that is 30 minutes.
- Do not use the light glasses in traffic or in other conditions that require good visibility.
- The light glasses must not be used in a damp or wet environment.

## Before use

Before using the glasses, charge the battery first.

1. Insert the micro USB cable into the device.
2. Connect the other end directly to a PC or an adapter and connect it to the power supply.
3. Charge the glasses until the battery indicator (next to the on/off button) indicates that the battery is fully charged. If the battery is completely discharged, charging takes approximately 2 hours. Thereafter, the period of use is approx. 2 hours; it may be somewhat shorter at the beginning.

## Contents of the package

The medisana DG 100 Light Glasses are supplied in a complete package with a USB cable, microfibre case and cleaning cloth, light blue lenses and stylish frame with integrated LEDs. On the inside of the frame of the glasses you will find the on/off button, the battery indicator light and the USB port.

## Using the glasses

Use the blue-light glasses if you have been prescribed blue light. If you wish, you can also use the light glasses without lenses. The lenses are removable and equipped with UV protection.

## Battery

To maximize battery life, we recommend turning off the light glasses when not in use. Extremely cold temperatures can affect the duration of the battery negatively. To maximize battery duration in cold weather conditions, the glasses should be stored in a warm place (room temperature) or be put in a warm place before use.

## Manual operation

The table gives an overview of the use of light glasses.

Function	Operation	Result
Battery check	Press the on/off button for 2 seconds	The battery indicator lights illuminate (when the device is charged)
Turn on the glasses	Press the on/off button for 1 second	The blue light comes on
Switch off the glasses	1 x briefly press the on/off button	The blue light goes off, the glasses are turned off

After 30 minutes, the light glasses turn off by themselves

The DG 100 Light Glasses are a multifunctional device and a complete solution for your health, whether you are travelling or at home. You can use the light glasses as an energy boost, against winterblues (seasonal fatigue), to improve your sleep, to avoid jet lag or to compensate for the negative effects of shift work.

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