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Hersteller Manufacturer

GB Safety instructions

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

- Use the device only according to its intended purpose as specified in the instruction manual.
- The warranty will be invalidated if the device is used for purposes other than those for which it is intended.
- Only use the infrared bulb with the infrared lamp for which it is intended.
- Do not use the device if it is not working correctly, if it has been dropped or has fallen into water or has been damaged.
- Do not use the device if the cable or plug are damaged.
- Keep the device and cable well away from hot surfaces. Do not place the device near sources of heat.
- Do not attempt to repair the device yourself in the event of a malfunction since this will invalidate the warranty. Repairs should only be carried out by authorised service centres.
- The device is designed for private households only and not for commercial use.
- Before using make sure that the connecting cable is not damaged. It must not come into contact with the hot electric light bulb.
- Before connecting the device to your power supply, please ensure that the supply voltage stated on the rating plate is compatible with your mains supply.
- Place device on a firm, level surface during use so that it cannot tip over. Do not attach device to walls or ceilings.
- Do not leave the device unattended when it is switched on.
- Keep flammable objects such as textiles, glasses and combs away from the infrared lamp.
- Keep the infrared lamp at least 80 cm away from flammable objects.
- Do not use the device unless it is completely dry.
- Do not use the device outdoors or in damp rooms.
- The device must not be used in the immediate vicinity of bathtubs, showers or washbasins filled with water, and must not be used with wet hands.
- Protect the device from moisture.
- Do not immerse the device in water or other fluids.
- Never carry, pull or rotate the unit by the mains cable and never pinch the cable between other objects.
- Do not cover the device when it is warm.
- Protect the device from shocks and vibration.
- **Attention, scalding danger!**
 - The bulb becomes hot.
 - Do not touch the infra-red bulb and do not remove with the power cord plugged in.
 - Always disconnect the device from the power supply and allow the lamp to cool before touching it.
- Always switch the device off after use, before cleaning and care and in the event of malfunctions during operation by disconnecting the plug from the socket.
- Always the infrared lamp to cool before putting it away.
- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children shall not play with the appliance.
- Cleaning and user maintenance shall not be made by children without supervision.
- Pain-killers and relaxants reduce the sensitivity of the skin to pain and heat; the same applies to the consumption of alcohol. Consult your doctor before using the device.
- Persons, whose sensitivity is disturbed or impaired may only apply the device according to the instructions of a physician.
- This device is unsuitable for persons who are insensitive to heat.
- Do not treat any parts of the body that have swellings, burns, inflammation, peeling skin, wounds or other sensitive points.
- Do not use the device on skin that feels painful or numb.
- If the skin becomes irritated, discontinue the treatment and consult a physician.
- Always seek the advice of a doctor in case of doubt and follow his instructions for treatment.
- The distance to the lamp must be at least 40 cm to avoid that the intensity of radiation gets too high and the warmth is felt as painful.
- Do not look directly into the infra-red light. Always close your eyes during radiation in order to prevent retinal injuries. You can also cover your eyes with a swab.
- Do not exceed the maximum radiation time of 15 minutes.
- The treatment should be pleasant. Stop at once and consult your doctor if you feel pain or if the treatment seems unpleasant.

Device and controls

- 1 Infra-red light bulb
- 2 Lamp base
- 3 Power cord with plug
- 4 Lamp housing (adjustable in 4 directions)

Scope of supply

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use the it and contact your dealer or your service centre. The following parts are included:

- 1 **MEDISANA** infrared lamp **IR 100**
- 1 Instruction manual

If you notice any transport damage during unpacking, please contact your dealer without delay.

WARNING
Please ensure that polythene packing is kept away from the reach of children! Risk of suffocation!

Cosmetic treatment

The **MEDISANA** infrared lamp **IR 100** is suitable for assisting facial and beauty care, particularly in the case of bad skin. Remove all traces of make-up before treatment. The effect of the infra-red radiation opens up the pores in the skin. This results in better and more effective cleaning of the skin. Creams and other cosmetic materials are absorbed more intensively and quickly by the skin.

Operation

- Please follow the safety instructions.
- Check the power cord 3.
- Place the device on its base 2 on a firm, level surface.
- The infrared bulb 1 must be firmly screwed in.
- The device has an adjustable lamp housing 4. The 4 settings from 0 ° to 45 ° allow the device to be individually positioned in accordance with requirements.
- To prevent burns, only touch the lamp by the base if you wish to change the position.
- Plug the device into the mains and it will start up immediately.
- The effect of the radiation is greatest from a short distance. Please operate the equipment at a distance at which the heat feels comfortable, but no less than 40 cm. Increase the distance if the heat feels uncomfortable.
- The duration of use is oriented to your sensitivity, but do not use for more than 15 minutes.
- Unplug the equipment from the mains when treatment is complete.

WARNING
Do not exceed a radiation time of 15 minutes and 2 sessions a day. The infra-red radiation emitted by the device is not suitable for tanning.

Cleaning and maintenance

- Before cleaning and replacing the bulb of the infrared lamp, unplug the equipment from the mains and allow the device to cool.
- Clean the device after cooling, 4 and 2, with a damp cloth and then rub dry.
- Do not use aggressive cleaning and scouring materials.
- Never immerse the device in water.
- Please ensure that water does not penetrate the internals of the device. Do not use the device again until it is completely dry.
- Always replace the bulb of the infra-red lamp with the same bulb type.
- Store the device in a dry, cool place.

Infra-red light bulb 1

It must not be used one with a higher power as indicated on the rating plate. Replacements for the light bulb can be bought where the appliance has been bought or ordered directly from the producer (indicate model!). The following replacement for the light bulb has to be used: Philips Infrared R 95E 100 W.

In case the glass of the light bulb is broken immediately disconnect the appliance. **Attention! Injuring danger due to splinters of glass.** Let the appliance cool down. Carefully remove the part of the light bulb which is screwed in the socket with a pair of pincers (protect your eyes) or have the appliance repaired by a service agent. The light bulb is not part of the guarantee, it won't be replaced free of charge.

Disposal

This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Consult your municipal authority or your dealer for information about disposal.

Technical specifications

Name and model : **MEDISANA** Infrared lamp **IR 100**
 Current supply : 220 - 240 V~ 50 - 60 Hz
 Consumption : 100 watts
 Dimensions LxWxH : approx. 16.5 x 13.5 x 20.5 cm
 Weight : approx. 0.36 kg
 Storage conditions : clean and dry
 Item number : 88232
 EAN number : 40 15588 88232 6
 Spare bulb : Item number 30101

In accordance with our policy of continual product improvement, we reserve the right to make technical and visual changes without notice.

The current version of this instruction manual can be found under www.medisana.com

Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is. The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

MEDISANA GmbH
 Jagenbergstraße 19, 41468 NEUSS
 GERMANY

For service, accessories and spare parts, please contact:

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