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# IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

## 1 Safety Information !

- Use the device only according to its intended purpose as specified in the instruction manual.
- The warranty will be invalidated if the device is used for purposes other than those for which it is intended.
- The scale has been manufactured for domestic use. It is not suitable for commercial use in hospitals or other medical institutions.
- Any treatment or diet for people who are underweight or overweight requires the qualified advice of a professional (doctor, dietician). The values determined with the scales may be source of useful support.
- This device is not suitable for pregnant women!
- These scales are not suitable for people with pacemakers or other medical implants. The body fat information may be inaccurate in people with diabetes or other medical/physical limitations. The same also applies to very highly trained athletes.
- This device is not designed to be used by persons (including children) with limited physical, sensory or mental abilities, or by persons with insufficient experience and/or knowledge, unless under observation by a person responsible for their safety, or unless they have been instructed in the use of the device.
- Do not use the device if it is not working correctly, if it has been dropped or has fallen into water or has been damaged.
- Do not use the scale in a location where extremely high temperatures or humidity may occur.

- Protect the unit against moisture. Keep the scale away from water. The surface of the scales is slippery when wet. Keep the surface dry.

• ***Risk of tipping over!***

- Do not place the scale on an uneven surface. Do not stand on the corners/edges of the scale. When weighing yourself, do not stand on one side or the edge of the scales.

• ***Risk of slipping!***

- Do not stand on the scale with wet feet. Do not stand on the scale wearing socks.

- Place the scales on a firm, level base. Soft, uneven surfaces are unsuitable for the measuring process and lead to incorrect results.

- The scales have a measuring range up to 180 kg (396 lbs / 28 st 4 lbs). Do not overload the scales – this can cause irreversible damage.
- 
- Handle the scale with care. The scale must not be subjected to impacts or vibration. Do not fall or drop anything on the scales.
- Do not attempt to dismantle the scale, otherwise the warranty will be invalidated. The scale does not contain any parts that need to be maintained by the user, with the exception of the battery.
  - Do not attempt to repair the device yourself in the event of a malfunction since this will invalidate the warranty. Repairs should only be carried out by authorised service centres.

## Thank you for your confidence in us and congratulations!

With the Personal scales with body analysis function **BS 475** you have acquired a quality product from **MEDISANA**. This unit is intended for weighing and calculating the proportion of bodyfat, water content and muscle mass in people. To ensure that your Personal scales with body analysis function **BS 475** provides successful results for a long time, we recommend that you read carefully through the following notes on their use and care.

## 2 Items supplied and packaging

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use the appliance and contact your supplier or your service centre.

The following parts are included:

- 1 **MEDISANA** Personal scales with body analysis function **BS 475**
- 2 Lithium coin cell batteries, 3 V / type CR2032
- 1 Instruction manual

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your supplier without delay.

### **WARNING**

**Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!**

## 3 Performance features

• Convenient „step-on“ switch • Top quality safety glass (6 mm) • Sensor technology • Age entry from 10 to 80 years • Height entry from 100 to 240 cm • Measuring range to 180 kg, 396 lb or 28st 4lb • Graduation 100 g, 0,2 lb or 1/4 lb • Switchable between KG, LB and ST • Direct measurement of body weight • Easy to use • Automatic switch-off • Overload warning • Low battery warning

## 4 Prerequisites for correct measuring results

**Reliable measurements can only be achieved under certain prerequisites:**

- Always measure barefoot.
- The feet must have a good contact with the electrodes on the standing area.
- Stand upright.
- Carry out the measurement approximately 15 minutes after getting up, or after bathing or showering, with towel-dried feet.
- Please measure at the same time of day if possible.
- Do not measure after strenuous physical exercise. Wait at least an hour before measuring.
- There must not be any dirt stuck to the soles of your feet. Erroneous measuring results may occur if the soles of your feet are dry or extremely callused.

The amount of body fat, water content and muscle mass can only be reliably monitored using the measurements if they are taken consistently. Measurements that are taken under varying conditions can return different values.

## 5 Insert/replace batteries

Two button cell batteries (type CR2032) are already inserted in your new scale. The insulating strip must be pulled out of the battery compartment **1** on the underside of the unit before the scale can be operated. Replace the battery if the battery change symbol "**LO**" appears in the display **2** or if nothing appears in the display after you have stepped on the device. To do this, open the battery compartment **1** beneath the device and insert two cell batteries (type CR2032). Please note polarity (positive pole upwards; facing towards the lid). Close the battery compartment **1** again.

### **WARNING** *Battery safety information*

- Do not disassemble batteries!
- Clean the battery and device contacts if necessary before putting in the batteries!
- Remove discharged batteries from the device immediately!
- Increased risk of leakage, avoid contact with skin, eyes and mucous membranes! If battery acid comes in contact with any of these parts, rinse the affected area with copious amounts of fresh water and seek medical attention immediately!
- If a battery has been swallowed seek medical attention immediately!
- Insert the batteries correctly, observing the polarity!
- Keep the battery compartment well sealed!
- Remove the batteries from the device if it is not going to be used for an extended period!
- Keep batteries out of children's reach!
- Do not attempt to recharge these batteries! ***There is a danger of explosion!***
- Do not short circuit! ***There is a danger of explosion!***
- Do not throw into a fire! ***There is a danger of explosion!***
- Keep unused batteries in their packaging away from metal objects in order to prevent short circuiting!
- Do not throw used batteries into the household refuse; put them in a hazardous waste container or take them to a battery collection point, at the shop where they were purchased!

## 6 Change the unit of weight

You may have your weight displayed in kilograms (kg), pounds (lb) or stones (st). Switch the scales on by touching with the foot and press the **3** button until the desired units are displayed.

## 7 Weighing "Step-on"-function

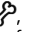
1. Make sure the scales are on a firm, level surface. If you only want to weigh yourself but have moved the scales just before, you must first initialise the scales. To do this, press your foot on the middle of the surface of the scales. "**0.0 kg**" appears in the display. **When the scales switch off, they are ready for the "step-on" function.** This procedure is not necessary if you have not moved the scales.
2. Step onto the scales and stand still. The device switches on automatically and the display **2** briefly shows "**0.0 kg**".
3. Your weight is measured, flashes three times and is then finally displayed.
4. Step down from the scales. The scales switch off automatically after 12 seconds.

## 8 Programming personal data

The scales can store data for up to 10 people: sex, age, height. The scales have preset values. These are: **kg, male, 30 years old, 170 cm.**

1. Press the SET **6** button to switch on the scales. The memory location is flashing in the display.
2. Now press **▲ 5-** or **▼ 4-** button to select the desired memory location (0 - 9) for the user profile.
3. Press the SET-button to confirm the selection. Now the symbol for the sex is flashing in the display.  
 = Man     = Woman
4. Set your sex by pressing the **▲ 5-** or **▼ 4-** button to select the appropriate symbol in the display.
5. Press the SET **6** button to save the setting. The preset age starts flashing in the display.
6. Select your age using the **▲ 5-** and **▼ 4-** buttons.
7. Press the SET **6** button to save the setting. The preset height starts flashing in the display.
8. Select your height using the **▲ 5-** and **▼ 4-** buttons.
9. Press the SET **6** button to save the setting.
10. The settings are now concluded. In the display "0.0" is shown.

## 9 Weighing and measuring body fat, water content and muscle mass

1. Press the SET **6** button to switch on the scales. The memory location flashes in the display.
2. By pressing the DOWN- and UP - buttons you may choose the memory location (0 - 9).
3. Your personal data is displayed.
4. Wait until "0.0" appears on the display. Step barefoot onto the scales and stand still. Place your feet on the electrodes **7**. First, your weight is displayed.
5. The values measured for body fat **fat%**, body water **TBW**, muscle mass , bones mass value , BMI **BMI** and calorific requirement are shown one after another in the display. The series of measured values is repeated twice in the display. The scales then switch off automatically. Step off the scales.

### NOTE

The data in the following tables is for orientation purposes only. If you have any questions about your proportion of body fat, please consult your doctor.

### Body fat proportion range in %

*female*

Age	low Underfat	normal Health	high Overfat	very high Obese
<b>12-20</b>	<18	18-28	28-33	>33
<b>21-42</b>	<20	20-30	30-35	>35
<b>43-65</b>	<21	21-31	31-36	>36
<b>66-100</b>	<22	22-32	32-37	>37

*male*

Age	low Underfat	normal Health	high Overfat	very high Obese
<b>12-20</b>	<15	15-21	21-26	>26
<b>21-42</b>	<17	17-23	23-28	>28
<b>43-65</b>	<18	18-24	24-29	>29
<b>66-100</b>	<19	19-25	25-30	>30

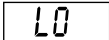
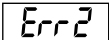
### The Body-Mass-Index (BMI)

*same for  
female  
and  
male*

Age	low	normal	high
<b>&lt;24</b>	<19	19-24	>24
<b>25-34</b>	<20	20-25	>25
<b>35-44</b>	<21	21-26	>26
<b>45-54</b>	<22	22-27	>27
<b>55-64</b>	<23	23-28	>28
<b>&gt;64</b>	<24	24-29	>29

## 10 Error messages

The following error messages may appear in the display:

	The scale is overloaded.
	The battery is low and must be replaced.
	Body analysis values cannot be calculated. See point 4 (Prerequisites for correct measuring results).

If your personal scales do not function to your satisfaction, check the following before contacting a service centre.

- Check whether the batteries have been inserted correctly.
- Check whether the correct unit of weight has been selected.
- Check whether the scales are freestanding on a solid and even surface. They must not be touching a wall or any other object.
- Weigh yourself again.

## 11 Care and maintenance

Remove the battery before cleaning the unit. Never use aggressive cleaning agents or stiff brushes. Clean the scale using a soft and lightly moistened cloth. Never use abrasive cleaning agents or alcohol. Do not allow any water to get into the unit. Do not use the unit again until it is completely dry. Do not expose the unit to direct sunlight; protect it against dirt and moisture. Do not store the scale in a vertical position, and remove the batterie from the unit if it will be unused for a longish period. Otherwise there is a risk that the battery might leak.

## 12 Disposal

This product must not be disposed of together with domestic waste.

All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Please remove the battery before disposing of the device/unit. Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop.

Consult your municipal authority or your supplier for information about disposal.

## 13 Technical specifications

Name and model	: <b>MEDISANA</b> Personal scales with body analysis function <b>BS 475</b>
Voltage supply	: 6 V = , 2 x 3 V Lithium coin cell batteries/ type CR2032
Display system	: Digital display
Memory	: for 10 persons
Measuring range	: up to 180 kg, 396 lb or 28 st 4 lb
Maximum measuring deviation	: $\pm 1$ %
Body fat	: 5 - 50 % in steps of 0.1 %
Body water	: 20 - 70 % in steps of 0.1 %
Muscle mass	: 20 - 70 % in steps of 0.1 %
Bone mass	: 0,5 - 8 kg in steps of 0.1 kg
Graduation	: 100 g, 0,2 lb or 1/4 lb
Autom. switch-off	: after approx. 12 seconds
Dimensions (L x W x H)	: 30 x 30 x 2,4 cm
Weight approx.	: 1,7 kg
Item number	: 40432
EAN number	: 40 15588 40432 0

**In accordance with our policy of continual product improvement, we reserve the right to make technical and visual changes without notice.**

## 14 Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be corrected free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
  - a. All damage which has arisen due to improper treatment, e.g. nonobservance of the user instructions.
  - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
  - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
  - d. Accessories which are subject to normal wear and tear (batteries etc.).
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

### **MEDISANA AG**

Jagenbergstraße 19  
41468 NEUSS  
GERMANY

E-Mail: [info@medisana.de](mailto:info@medisana.de)

Internet: [www.medisana.de](http://www.medisana.de)

The service centre address is shown on the attached leaflet.

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